

Instructions for use

MASTERmatic LUX M25 L - 1.009.3630

MASTERmatic LUX M05 L - 1.009.3640

www.goldident.ru



KaVo. Dental Excellence.

Distributed by:

KaVo Dental GmbH
Bismarckring 39
D-88400 Biberach
Phone +49 7351 56-0
Fax +49 7351 56-1488

Manufacturer:

Kaltenbach & Voigt GmbH
Bismarckring 39
D-88400 Biberach
www.kavo.com



Contents

1	User instructions.....	6
2	Safety.....	8
2.1	Description of safety instructions.....	8
2.1.1	Warning symbol.....	8
2.1.2	Structure.....	8
2.1.3	Description of danger levels.....	9
2.2	Safety instructions.....	11
3	Product description.....	20
3.1	Purpose – Intended use.....	22
3.2	Technical data.....	24
3.3	Transportation and storage conditions.....	25
4	Start up and shut down.....	27
4.1	Checking the amount of water.....	29
5	Operation.....	31
5.1	Attach the medical device.....	31

5.2	Remove the medical device.....	34
5.3	Insert the milling cutters or diamond grinders.....	35
5.4	Removing the milling tool or diamond grinder.....	39
6	Troubleshooting.....	41
6.1	Check for malfunctions.....	41
6.2	Troubleshooting.....	44
6.2.1	Replacing the O-rings.....	44
6.2.2	Cleaning the spray nozzle.....	45
6.2.3	Change the water filter.....	46
7	Reprocessing steps in accordance with ISO 17664.....	47
7.1	Preparations at the site of use.....	47
7.2	Cleaning.....	48
7.2.1	Manual cleaning - external.....	48
7.2.2	Automated external cleaning.....	49
7.2.3	Manual cleaning of the inside.....	50
7.2.4	Automated internal cleaning	52

7.3	Disinfection.....	53
7.3.1	Manual disinfection - external.....	56
7.3.2	Manual disinfection - internal.....	57
7.3.3	Machine disinfection - external and internal.....	58
7.4	Drying.....	59
7.5	Care products and systems - Servicing.....	60
7.5.1	Servicing with KaVo Spray.....	62
7.5.2	Care of the KaVo SPRAYrotor.....	63
7.5.3	Servicing with KaVo QUATTROcare 2104 / 2104A.....	64
7.5.4	Servicing with KaVo QUATTROcare PLUS 2124 A.....	67
7.5.5	Servicing with KaVo QUATTROcare 2140 A.....	70
7.6	Packaging.....	72
7.7	Sterilisation.....	73
7.8	Storage.....	75
8	Tools and consumables.....	76
9	Terms and conditions of warranty.....	78



1 User instructions





Dear User

Congratulations on purchasing this KaVo quality product. By following the instructions below you will be able to work smoothly, economically and safely.

© Copyright by KaVo Dental GmbH

Symbols

	Refer to the chapter on Safety/Warning symbol
	Important information for users and service technicians

	Can be steam-sterilised at 134 °C -1 °C / +4 °C (273 °F -1.6 °F / +7.4 °F)
	Thermodisinfectable
	CE-Zeichen (Communauté Européenne). Ein Produkt mit diesem Zeichen entspricht den Anforderungen der anwendbaren EG-Richtlinie.
	Action request

Target group

This document is intended for dentists and their assistants. The section on starting up is also intended for service technicians.

2 Safety

2.1 Description of safety instructions

2.1.1 Warning symbol



Warning symbol

2.1.2 Structure



DANGER

The introduction describes the type and source of the hazard.

This section describes potential consequences of non-compliance.

- ▶ The optional step includes necessary measures for hazard prevention.
-

2.1.3 Description of danger levels

The safety instructions listed here, together with the three levels of danger will help avert property damage and injury.



CAUTION

CAUTION

indicates a hazardous situation that can cause damage to property or mild to moderate injuries.



WARNING

WARNING

indicates a hazardous situation that can lead to serious or fatal injury.



DANGER

DANGER

indicates a maximal hazard due to a situation that can directly cause death or fatal injury.

2.2 Safety instructions



WARNING

Hazard from incorrectly reconditioned products.

An infection hazard exists from contaminated products.

- ▶ Take suitable personal protective measures.

www.goldident.ru



WARNING

Hazards for the care provider and the patient.

In the case of damage, irregular running noise, excessive vibration, un-typical warming or when the cutter or grinder cannot be held.

- ▶ Do not use further and notify Service.



CAUTION

Burning hazard from hot instrument head and instruments cover.

If the instrument overheats, burns may arise in the oral area.

- ▶ Never contact soft tissue with the instrument head.



CAUTION

Hazard from use as a light probe.

Do not use the device as a light probe since the rotating cutters or grinders can cause injury.

- ▶ Use an appropriate light probe for additional illumination of the oral cavity or site of preparation.



CAUTION

Risk due to incorrectly stored instrument.

Damage to clamping system from dropping the instrument.

- ▶ After treatment, place the instrument properly in the cradle without the tool.

www.goldident.ru



CAUTION

Risks due to lack of control equipment.

Hazards can arise if control equipment is not available for changing the speed and the direction of rotation.

- ▶ The dental treatment unit connected must have control equipment for changing the speed and direction of rotation.
- ▶ A note is to be included in the documents accompanying the dental treatment unit, referring to responsibilities arising from safety, reliability and performance.
- ▶ The medical device may only be combined with a treatment centre released by KaVo.

**⚠ CAUTION**

Premature wear and malfunctioning from improper storage during long periods of nonuse.

Reduced product life.

- ▶ The medical device should be cleaned, serviced and stored in a dry location, according to instructions, before long periods of nonuse.



CAUTION

Hazard from the use of handpieces equipped with electronic micromotors.

Electronic micromotors generate much more energy than conventional pneumatic turbines and motors. Given the higher torque and speed, handpieces that are poorly serviced, damaged or used improperly can overheat which can cause serious burn injuries to the patient.

- ▶ Observe the following points.

The following guidelines must be observed to ensure safe use of the electrically driven handpieces:

- The service instructions for handpieces must be precisely followed when using KAVO spray or QUATTROcare care systems.

- Before each use, the handpiece must be checked for external damage.
- Before each use, perform a test run with the handpiece, and watch for atypical heating and unusual noise and vibration.
- Immediately stop using handpieces that act unusual.
- Never press the pushbutton during operation. This also includes lifting the cheek or tongue!



Note

For safety reasons, we recommend that the tool holder system be checked annually after the warranty period expires.


The following individuals are authorized to repair and service KaVo products:

- Technicians at KaVo branches throughout the world
- Technicians specially trained by KaVo


To ensure proper function, the medical device must be set up according to the reprocessing methods described in the KaVo Instructions for Use, and the care products and care systems described therein must be used. KaVo recommends specifying a service interval at the dental office for a licensed shop to clean, service and check the functioning of the medical device. This service interval depends on the frequency of use and should be adjusted accordingly.

Service may only be carried out by KaVo-trained repair shops using original KaVo replacement parts.

3 Product description

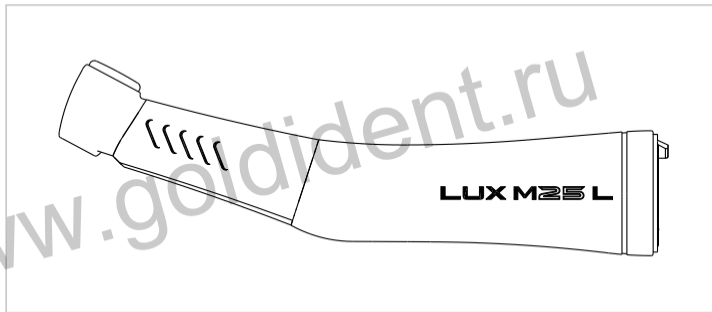

MASTERmatic LUX M25 L (Mat. no. 1.009.3630)

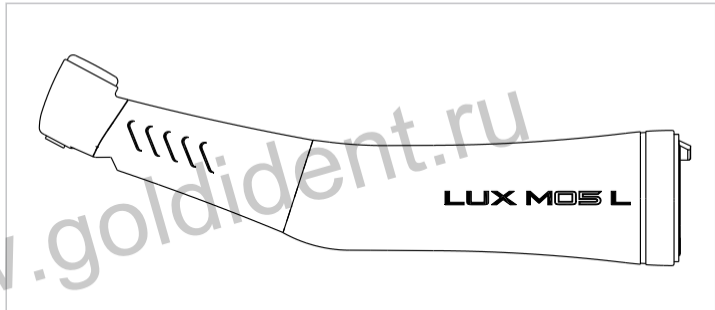

MASTERmatic LUX M05 L (Mat. no. 1.009.3640)

3.1 Purpose – Intended use

Indications for use:

This medical device is

- intended for dental treatment only. All other types of use or modifications of the product are not permitted and can be hazardous. The medical device is intended for the following applications: Removal of carious material, cavity and crown preparations, removal of fillings, processing of tooth and restoration surfaces.
- A medical device according to relevant national statutory regulations.

Proper use:

According to these regulations, this medical device may only be used for the described application by a knowledgeable user. You need to comply with the following:

- the applicable health and safety regulations
- the applicable accident prevention regulations
- these Instructions for use

According to these regulations, the user is required to:

- only use equipment that is operating correctly,
- adhere to the specified intended use
- protect him or herself, the patient and third parties from danger, and
- avoid contamination from the product.

3.2 Technical data

Drive speed	max. 40,000 rpm
Speed transmission	1 : 5
Identification	1 red ring
Spray water pressure	0.8 to 2.0 bar (12 to 29 psi)
Spray air pressure	1.0 to 2.0 bar (15 to 29 psi)
Spray air quantity	min. 1.5 NI/min (at 2 bar)
Cooling air flow	5.5 to 9.5 NI/min
Pushbutton chuck	Ø 1.6 mm

Usable with contra-angle cutters or grinders.

The contra-angle handpiece can be mounted on all INTRAmatic (LUX) motors and motors with a connector in accordance with ISO 3964 / DIN 13940.

3.3 Transportation and storage conditions





CAUTION



It is hazardous to start up the medical device after it has been stored strongly refrigerated.

This can cause the medical device to malfunction.

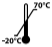- ▶ Prior to start-up, very cold products must be heated to a temperature of 20°C to 25°C (68°F to 77°F).

	Temperature: -20°C to +70°C (-4°F to +158°F)
	Relative humidity: 5% RH to 95% RH absence of condensation
	Air pressure: 700 hPa to 1060 hPa (10 psi to 15 psi)
	Protect from moisture

4 Start up and shut down



WARNING

Hazard from non-sterile products.

Infection hazard for care provider and patient.

- ▶ Before first use and after each use, prepare and sterilise the medical device and accessories accordingly.



WARNING

Disposal of the product in the appropriate manner.

Prior to disposal, the product and accessories must be appropriately prepared or sterilised if this is necessary.

**⚠ CAUTION****Damage from contaminated and moist cooling air.**

Contaminated and moist cooling air can cause malfunctions.

- ▶ Make sure that the supply of cooling air is dry, clean, and uncontaminated according to EN ISO 7494-2.

4.1 Checking the amount of water


CAUTION

Overheating of the tooth due to insufficient amount of cooling water.

Insufficient spray water can cause the medical device to overheat and damage the pulp and tooth.



- ▶ Adjust the water amount for the spray cooling to a minimum of 50 ml/min (3.1 inch³).
- ▶ Check spray water channels and if necessary clean spray nozzles with the nozzle needle (**Mat. no. 0.410.0921**).
- ▶ Check water filter and replace, if necessary.

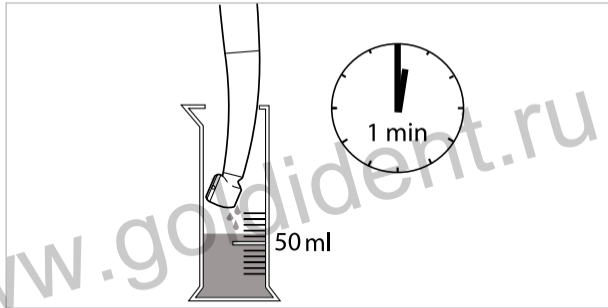

5 Operation

5.1 Attach the medical device



WARNING

Release of the medical device during treatment.

A medical device that is not properly locked in place can release from the motor coupling and fall off.

- ▶ Carefully pull on it before each treatment to ensure that the medical device is securely locked on the motor coupling.



CAUTION

Connect to the drive motor.

Contra-angle handpiece blocked.

- ▶ Only start the contra-angle handpiece when the chuck is closed.



CAUTION

Removing and attaching the contra-angle handpiece while the drive motor is rotating.

Damage to the drive.

- ▶ Never attach or remove the contra-angle handpiece while the drive motor is rotating!




CAUTION

Damage

If the footswitch is pressed while you are connecting and removing the medical device, you can damage the medical device and motor coupling.

- ▶ Do not connect or remove the medical device while pressing the foot-switch.
-
- ▶ Lightly spray O-rings on motor coupling with Multifunction syringe Spray.

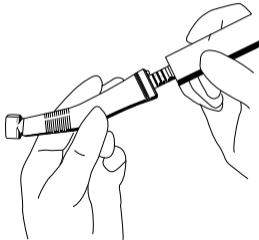

- ▶ Attach the medical device to the motor coupling and turn it until the guide stud audibly snaps into place.

- ▶ Pull on the medical device to make sure that it is securely affixed to the coupling.

5.2 Remove the medical device

- ▶ Unlock the medical device from the motor coupling by twisting it slightly and then pulling it along its axis.

5.3 Insert the milling cutters or diamond grinders

Note

Only use carbide cutters or diamond grinders that comply with EN ISO 1797-1 type 3, are made of steel or hard metal and meet the following criteria:

- Shaft diameter: 1.59 to 1.60 mm
- Overall length M25 L: max. 25 mm
- Shaft clamping length M25 L: at least 11 mm
- Overall length M05 L: max. 19 mm
- Shaft clamping length M05 L: at least 9 mm
- Blade diameter M25 L and M05 L: max. 2 mm





CAUTION

Use of unauthorised cutters or grinders.

Injury to the patient or damage to the medical device.

- ▶ Observe the instructions for use and use the cutter or grinder properly.
- ▶ Only use cutters or grinders that do not deviate from the specified data.



CAUTION

Injury from using worn drill bits or burs.

Drill bits or burs could fall out during treatment and injure the patient.

- ▶ Never use drill bits or burs with worn shafts.

**⚠ CAUTION**

Injury hazard from cutters or grinders.

Infections or cuts.

- ▶ Wear gloves or fingerstalls.

www.goldident.ru



CAUTION

Hazard from defective chucking system.

The cutter or grinder could fall out and cause injury.

- ▶ Pull on the cutter or grinder to check that the chucking system is okay and the cutter or grinder is securely held. When checking, inserting and removing, use gloves or a fingerstall to prevent an injury or infection.



- ▶ Forcefully press the push button with your thumb and simultaneously insert the cutter or grinder all the way.

- ▶ Check that the cutter or grinder is securely attached by pulling on it.

5.4 Removing the milling tool or diamond grinder

WARNING

Hazard from rotating cutter or grinder.

Lacerations and damage to the chucking system.

- ▶ Do not touch rotating cutter or grinder.!
- ▶ Never press the press-button while the cutter or grinder is rotating!
- ▶ Remove the cutter or grinder from the contra-angle handpiece after treatment to avoid injury or infection while storing it.





- ▶ After the cutter or grinder has stopped rotating, press the press-button with your thumb and simultaneously pull out the drill bit or bur.

www.goldident.ru

6 Troubleshooting

6.1 Check for malfunctions



CAUTION

Missing or damaged O-rings.

Malfunctions and premature failure.

- ▶ Make sure that all O-rings are on the coupling and undamaged.



CAUTION

Heating of the product.

Burns or product damage from overheating.

- ▶ Do not use the product if it is irregularly heated.

-
- ▶ The medical device is too hot while idling:

Check the amount of cooling air.

- ▶ The medical device is too hot while working:

Caring for the medical device.

- ▶ When the speed drops or is uneven:
Caring for the medical device.
- ▶ An O-ring is missing on the motor coupling:
Replace O-ring.

www.goldident.ru

6.2 Troubleshooting

6.2.1 Replacing the O-rings



CAUTION

Hazard from improper care of the O-rings.

Malfunctions or complete failure of the medical device.

- ▶ Do not use Vaseline or other grease or oil.



Note

The O-ring on the coupling may only be lubricated with a cotton ball wet with KAVO spray.

- ▶ Press the O-ring between your fingers to form a loop.
- ▶ Shove the O-ring to the front, and remove it.
- ▶ Insert new O-rings into the grooves.

6.2.2 Cleaning the spray nozzle.

CAUTION

Hazard from insufficient spray water.


Overheating of the medical device and damage to the tooth.

- ▶ Check the spray water channels and clean the spray nozzles with the nozzle needle **Mat. no. 0.410.0921** if necessary.
- ▶ Check the water filter and exchange if necessary.



- ▶ Use the nozzle needle (**Mat. no. 0.410.0921**) to free the water passage in the spray nozzles.

6.2.3 Change the water filter

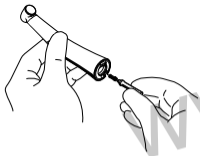

- ▶ Screw out and remove the filter with the wrench (**Mat. no. 1.002.0321**).
- ▶ Insert the new filter (**Mat. no. 1.002.0271**) and screw it in with the wrench.

7 Reprocessing steps in accordance with ISO 17664

7.1 Preparations at the site of use



WARNING

Hazard from nonsterile products.

There is a risk of infection from contaminated medical devices.

- ▶ Take suitable personal protective measures.

- ▶ Remove all residual cement, composite or blood immediately.
- ▶ Reprocess the medical device as soon as possible after treatment.
- ▶ Remove the cutter or grinder from the medical device.

- ▶ The medical device must be dry when transported for reprocessing.
- ▶ Do not place it in solutions or the like.

7.2 Cleaning



CAUTION

Malfunctions from cleaning in the ultrasonic unit.


Defects in the product.

- ▶ Only clean manually or in a thermodisinfectator.

7.2.1 Manual cleaning - external

Accessories required:

- Tap water $30\text{ °C} \pm 5\text{ °C}$ ($86\text{ °F} \pm 10\text{ °F}$)

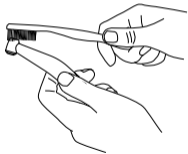

- Brush, e.g. medium-hard toothbrush
- ▶ Brush off under flowing tap water.

7.2.2 Automated external cleaning



KaVo recommends washer disinfectors in compliance with EN ISO 15883-1, which are operated using alkaline cleaning agents.

- ▶ For program settings as well as cleansers and disinfectants to be used, please refer to the Instructions for Use of the thermodisinfectant.

- ▶ In order to prevent negative effects on the medical device, make sure that the interior and the exterior of the medical device are dry after completion of the cycle, and then grease it immediately with servicing agents from the KaVo care system.

7.2.3 Manual cleaning of the inside

Validated interior cleaning (residual protein removal) can only be accomplished with KaVo CLEANspray and KaVo DRYspray.

- ▶ Cover the medical device with the KaVo CLEANpac bag, and place it on the corresponding care adapter. Press the spray button three times for 2 seconds each time. Remove the medical device from the spray attachment and let the cleaner work for one minute.
- ▶ Afterwards, rinse for 3-5 seconds with KaVo DRYspray.

See also:

 KaVo CLEANspray / KaVo DRYspray Instructions for Use**Note**

KaVo CLEANspray and KaVo DRYspray for manual interior cleaning are only available in the following countries:

Germany, Austria, Switzerland, Italy, Spain, Portugal, France, Luxembourg, Belgium, Netherlands, United Kingdom, Denmark, Sweden, Finland and Norway.

In other countries interior cleaning can only be carried out with thermodisinfectors in accordance with EN ISO 15883-1.

7.2.4 Automated internal cleaning



KaVo recommends washer disinfectors in compliance with EN ISO 15883-1, which are operated using alkaline cleaning agents.

- ▶ For programme settings as well as cleansers and disinfectants to be used, please refer to the Instructions for Use of the washer disinfectant.
- ▶ In order to prevent negative effects on the medical device, make sure that the interior and the exterior of the medical device are dry after completion of the cycle, and then grease it immediately with servicing agents from the KaVo care system.

7.3 Disinfection



WARNING

Hazard due to incomplete disinfection.

Principally, KaVo recommends carrying out an final disinfection of the un-packaged item if complete disinfection cannot be guaranteed without this measure.

www.goldident.ru



 **CAUTION**

Malfunctioning from using a disinfectant bath or disinfectant containing chlorine.

Defects in the product.

- ▶ Only disinfect in a thermodisinfecter or manually.

**⚠ CAUTION**

Malfunctioning from using the disinfectant bath or chlorine-containing disinfectants.

Defects in the device.

- ▶ Do not use an ultrasonic bath.

**⚠ CAUTION**

Never use alkaline or chlorine-containing disinfectants.

The saline solution corrodes the metal parts.

- ▶ Immediately remove all residue.

7.3.1 Manual disinfection - external



KaVo recommends the following products based on material compatibility.

- Mikrozyd AF made by Schülke & Mayr (liquid or cloths)
- FD 322 made by Dürr
- CaviGide made by Metrex

Consumables required:

- Cloths for wiping off the medical device.

- ▶ Spray the disinfectant on a cloth, then thoroughly wipe down the medical device and leave the disinfectant to soak in according to the instructions from the disinfectant manufacturer.
- ▶ Follow the instructions for use of the disinfectant.

7.3.2 Manual disinfection - internal

The efficacy of manual internal disinfection must be demonstrated by the manufacturer of the disinfection agent. With KaVo products, use only disinfection agents that have been released by KaVo with respect to the compatibility of materials (e.g. WL-cid / made by ALPRO).

- ▶ Blow off with compressed air until no water drops are visible.
- ▶ Immediately after internal disinfection, lubricate the KaVo medical device immediately with care agents from the KaVo care system.

7.3.3 Machine disinfection - external and internal



KaVo recommends washer disinfectors in compliance with EN ISO 15883-1, which are operated using alkaline cleaning agents.

- ▶ For programme settings as well as cleansers and disinfectants to be used, please refer to the Instructions for Use of the washer disinfector.
- ▶ In order to prevent negative effects on the medical device, make sure that the interior and the exterior of the medical device are dry after completion of the cycle, and then grease it immediately with servicing agents from the KaVo care system.

7.4 Drying

Manual Drying

- ▶ Blow off the outside and inside with compressed air until water drops are no longer visible.

Automatic Drying

The drying procedure is usually a part of the cleaning program of the thermodisinfectors.

- ▶ Follow the instructions for use of the thermodisinfectors.

7.5 Care products and systems - Servicing



WARNING

Sharp cutters or grinders in the medical device.

Risk of injury from sharp and/or pointed cutters or grinders.

- ▶ Remove cutter or grinder.

www.goldident.ru



CAUTION

Premature wear and malfunctions from improper servicing and care.

Reduced product life.

- ▶ Perform proper care regularly!




Note

KaVo only guarantees that its products will function properly when the care products are used that are listed as accessories since they were tested for proper use on our products.

7.5.1 Servicing with KaVo Spray

KaVo recommends servicing the product as part of the reprocessing after each use, i.e. after each cleaning, disinfection, and before each sterilisation.

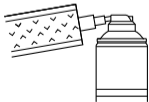

- ▶ Remove the cutter or grinder.
- ▶ Cover the product with the Cleanpac bag.
- ▶ Plug the product onto the cannula, and press the spray button for one second.


Care of chucking system

KaVo recommends cleaning and servicing the chuck system once a week.



- ▶ Remove the cutter or grinder, place the spray nipple tip in the opening and spray.
- ▶ Carry out the servicing according to the instructions in the section, "Care with KaVo Spray".

7.5.2 Care of the KaVo SPRAYrotor

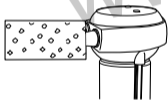

- ▶ Place the product on the appropriate coupling of the KaVo SPRAYrotor and cover it with a Cleanpac bag.

- ▶ Servicing the product.

See also:

- 📄 Instructions for use KaVo SPRAYrotor

7.5.3 Servicing with KaVo QUATTROcare 2104 / 2104A

Note

QUATTROcare 2104 / 2104 A is no longer included in the current delivery programme.


Follow-up products:

- ▶ QUATTROcare PLUS 2124 A
- ▶ QUATTROcare CLEAN 2140 A



Servicing and cleaning device with expansion pressure for the interior cleaning of inorganic residues and optimum care.

(no validated cleaning of the interior in accordance with German RKI requirements)

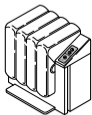

KaVo recommends servicing the product as part of the reprocessing after each use, i.e. after each cleaning, disinfection, and before each sterilisation.

- ▶ Remove the cutter or grinder.
- ▶ Servicing the product.

See also:

- 📄 Instructions for use KaVo QUATTROcare 2104 / 2104A

Servicing the clamping chuck

KaVo recommends cleaning and servicing the chuck system once a week.

See also:

📄 Instructions for use KaVo QUATTROcare 2104 / 2104A



- ▶ Remove the cutter or grinder, place the spray nipple tip in the opening and spray.
- ▶ Subsequently treat with the specified care products and systems.

See also:

📄 Servicing with KaVo QUATTROcare 2104 / 2104A

7.5.4 Servicing with KaVo QUATTROcare PLUS 2124 A

Servicing and cleaning device with expansion pressure for the interior cleaning of inorganic residues and optimum care.

(no validated cleaning of the interior in accordance with German RKI requirements)



- ▶ Remove the cutter or grinder.

- ▶ Servicing the product in the QUATTROcare PLUS.

See also:

- 📄 Instructions for Use KaVo QUATTROcare PLUS 2124 A

Servicing the clamping chuck

KaVo recommends cleaning and servicing the chuck system once a week.


See also:

- 📄 Instructions for Use KaVo QUATTROcare PLUS 2124 A

**Note**

Handpieces must be taken off the service couplings before the chuck service can be started and performed.

- ▶ Remove the service coupling chuck from the side hatch of the QUATROcare PLUS and attach it to coupling service point four, on the far right. A MULTIflex adaptor must be mounted there.

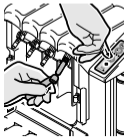

- ▶ Press the handpiece together with the guide bush of the chuck to be serviced against the tip of the service coupling.
- ▶ Press the button marked with the chuck service symbol.



Note

Close the chuck service mode.

Option 1: Place the dental handpieces in the QUATTROcare PLUS 2124 A, close the front door and start the service procedure.

Option 2: After three minutes with no service procedure running, the device automatically switches back to normal service mode.


See also:



Service with KaVo QUATTROcare PLUS

7.5.5 Servicing with KaVo QUATTROcare 2140 A

Programme-controlled cleaning and servicing device for perfect servicing of handpieces and turbines.

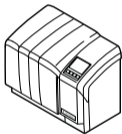

- ▶ Remove the cutter or grinder.
- ▶ Servicing the product in QUATTROcare CLEAN.

See also:

- 📄 Instructions for use KaVo QUATTROcare CLEAN 2140 A

Servicing the chuck

KaVo recommends cleaning and servicing the chuck system once a week using the collet servicing program integrated in the device.

See also:

 Instructions for use KaVo QUATTROcare CLEAN 2140 A

7.6 Packaging**Note**

The sterilisation bag must be large enough for the handpiece so that the bag is not stretched.

The quality and use of the sterilisation packaging must satisfy applicable standards and be suitable for the sterilisation procedure!

- ▶ Individually seal the medical device in the sterilised item packaging.

7.7 Sterilisation

Sterilisation in a steam steriliser (autoclave) in accordance with EN 13060 / ISO 17665-1

 **CAUTION**

Premature wear and malfunctions from improper servicing and care.

Reduced product life.

- ▶ Before each sterilisation cycle, service the medical device with KaVo care products.



CAUTION

Contact corrosion due to moisture.

Damage to product.

- ▶ Immediately remove the product from the steam steriliser after the sterilisation cycle!

The KaVo medical device has a maximum temperature resistance of up to 138 °C (280.4 °F).

Select a suitable procedure (depending on the available autoclave) from the following sterilisation processes:

- Autoclave with three times pre-vacuum:
 - at least 3 minutes at 134 °C -1 °C / +4 °C (273 °F -1.6 °F / +7.4 °F)

- Autoclave using the gravity method:
 - at least 10 minutes at 134 °C -1 °C / +4 °C (273 °F -1.6 °F / +7.4 °F)
 - ▶ Use according to the manufacturer's Instructions for Use.

7.8 Storage

Prepared products must be stored, protected from germs (as far as possible) and dust, in a dry, dark, cool room.



Note

Comply with the expiry date of the sterilised items.

8 Tools and consumables

Available from dental suppliers.

Material summary	Mat. no.
Instrument stand 2151	0.411.9501
Cellulose pad 100 units	0.411.9862
Cleanpac 10 units	0.411.9691
Nozzle needle	0.410.0921
Replacement filter	1.002.0271
Wrench	1.002.0321
O-ring	0.200.6120

Material summary	Mat. no.
KaVo CLEANspray 2110 P	1.007.0579
KaVo DRYspray 2117 P	1.007.0580
KaVo Spray 2112 A	0.411.9640
ROTAspray 2 2142 A	0.411.7520
QUATTROcare plus Spray 2140 P	1.005.4525

www.goldident.ru

9 Terms and conditions of warranty

The following warranty conditions apply to this KaVo medical device:

KaVo provides the end customer with a warranty of proper function and guarantees zero defects in respect of material and processing for a period of 24 months from the date of the invoice, subject to the following conditions:

In case of justified complaints, KaVo will honour its warranty with a free replacement or repair. Other claims of any nature whatsoever, in particular with respect to compensation, are excluded. In the event of default, gross negligence or intent, this shall only apply in the absence of mandatory legal regulations to the contrary.

KaVo shall not be liable for defects and their consequences that have arisen or may arise from natural wear, improper handling, cleaning or maintenance, non-compliance with operating, maintenance or connection instructions, calcination or corrosion, contaminated air or water supplies or chemi-

cal or electrical factors deemed abnormal or impermissible in accordance with KaVo's instructions for use or other manufacturer's instructions. The warranty granted does not usually extend to lamps, light conductors made of glass and glass fibres, glassware, rubber parts, and the colourfastness of plastic parts.

All liability is excluded if defects or their consequences originate from manipulations or changes to the product made by the customer or a third party that is not authorised by KaVo.

Warranty claims will only be accepted if the product is submitted along with proof of purchase in the form of a copy of the invoice or note of delivery. The dealer, purchase date, type, and serial number must be clearly evident from this document.

www.goldident.ru

www.goldident.ru

www.goldident.ru

www.goldident.ru



www.goldident.ru



KaVo. Dental Excellence.